Zorflex®

Accelerated Wound Healing™

Instructions for use

0120

Rev 04

Antimicrobial, conductive and odour absorbing primary dressing.

STERILE **8** Gamma irradiation sterilised. Do not use if pack is open or damaged. For single use only. Store in a dry place (<30°C / <86°F). Avoid exposure to temperatures above 50°C/122°F.

Description

Zorflex® dressing consists of a 100% pure activated carbon cloth. Zorflex® is a highly conformable dressing that follows the body contours to maintain contact with the wound surface. The Zorflex® dressing is black to grey in colour, variation in colour is possible, however this does not affect the performance. Zorflex® 100% pure activated carbon cloth provides an effective anti-microbial barrier to microbial contamination protecting the wound from invasive micro-organisms which may prevent the wound from healing. Zorflex® 100% pure activated carbon cloth is also effective against micro-organisms already present in the wound. The anti-microbial barrier properties of Zorflex® remain effective for a minimum of 7 days. Zorflex® has been shown to reduce odour, pain and exudation. Zorflex® is also conductive. Zorflex® can be used either dry or moistened with sterilised water.

Indications

Zorflex® is indicated as an antimicrobial dressing over discharging, partial and full thickness wounds such as: cavities and trauma, surgical sites, fungating carcinomas, pressure ulcers, venous ulcers, diabetic ulcers, donor and recipient graft sites.

Contra-indications

Not for use on dry wounds as this may cause the dressing to adhere to the wound.

Precautions

Use sterile water to cleanse, irrigate or moisten the wound. The use of oil based products, topical ointments or saline will reduce the effectiveness of Zorflex®. Do not reuse. If reused the performance of the product may deteriorate and cross contamination may occur. If the dressing has adhered to the wound soak it initially and take care not to damage underlying tissue upon removal.

Instructions for use

- Cleanse the wound with sterile water.
- Zorflex® will tear easily. To avoid fraying cut the dressing to shape, covering the periwound if maceration is evident.
- Without stretching the dressing, apply Zorflex® either way up to the wound surface.
- The dressing should be applied with the direction of stretch running along the limb to allow movement.
- Secure the Zorflex® dressing in place using an appropriate secondary dressing or compression therapy. Alternatively secure with a dressing retention sheet, tape or bandage. In the case of highly exuding wounds an absorbent secondary dressing is appropriate. Zorflex® allows transfer of the exudate into the secondary dressing which should be replaced as necessary without disturbing the Zorflex® or the wound bed.
- Should the dressing dry or adhere to the wound, moisten or soak the dressing with sterile water to assist removal and avoid disruption of the healing wound.
- Zorflex® can be left on the wound for up to 7 days, or as clinically indicated. Any threads or fibres which detach from the dressing can be easily seen and removed.
- Device should only be used as specified.
- Device should be used under relevant supervision by healthcare professionals.

Product Availability

Product Availability Code	Size	Availability
ZF05	5cm x 5cm (2" x 2")	Pouched only
ZF10	10cm x 10cm (4" x 4")	Pouched only
ZF15	15cm x 25cm (6" x 10")	Pouched only
ZF20	20cm x 10cm (8" x 4")	Pouched only

Manufacturer

Chemiviron Carbon Cloth Division, Rainton Bridge Industrial Estate
Houghton-le-Spring, Tyne & Wear, DH4 5PP, GB

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Zorflex® trade mark of Chemiviron Carbon.