



Independently doffing a leg stocking

REMOVING THE STEVE+ EASYON

Before taking off the stockings, you must remove the Steve + EasyON from the Steve+:

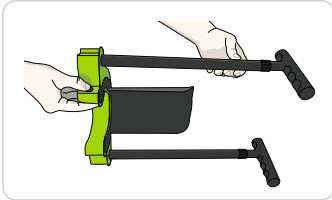

Grab the Steve+ in one hand, press one pushbutton with the other.

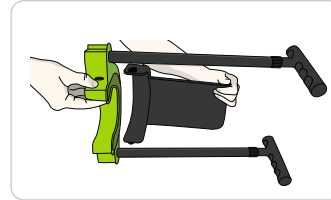


Pull the Steve+ EasyON out of the Steve+.


POSITION THE RING UNDER THE STOCKING

There are two ways to position the ring under the stocking:


1 PUSHING THE RING UNDER THE STOCKING

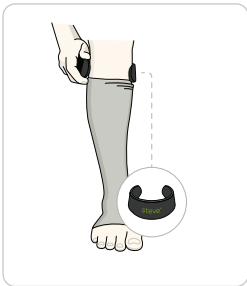

Place the ring around the leg above the stocking.

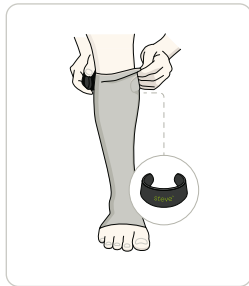


Slide one side of the ring a few inches/cm below the edge of the stocking.

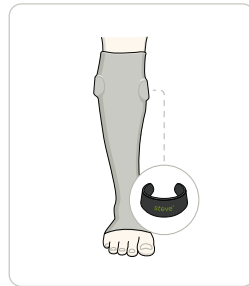


Then push the other side of the ring at parallel height under the stocking.

RIGHT RING POSITION

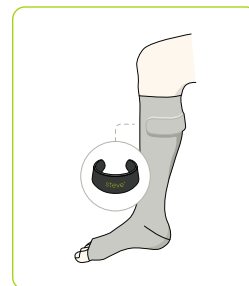

Without a silicone border, place the ring a few inches / cm below the edge.

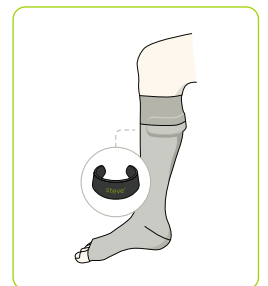


With a silicone border, place the ring half under the border.


2 PULL THE STOCKING OVER THE RING



Fold the stocking down a bit. Apply the ring around the leg.

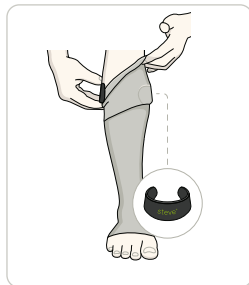


Pull one side of the stocking over the ring a few inches.

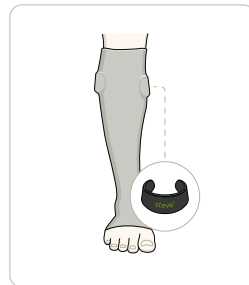

Grab the other side of the stocking and pull it parallel across the ring.

RIGHT RING POSITION

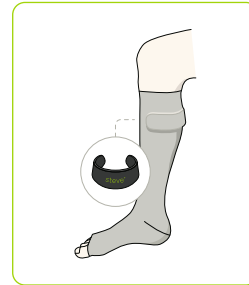

Without a silicone border, place the ring a few inches/centimeters below the edge.



With a silicone border, place the ring half under the border.