

# kliniderm® foam silicone

## Information for patients

### What is Kliniderm foam silicone?

Kliniderm foam silicone is soft, conformable foam dressing that is designed to manage fluid and creates a healing environment for your wound.


### How to apply and remove the dressing

Follow the instructions given to you by your healthcare professional.


### How often should the dressing be changed?

Your healthcare professional will advise how often you need to change your dressing.

See visual indicator below – please note this is a guide only.

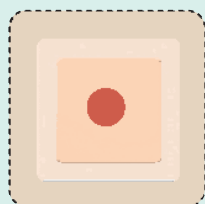

Dressing change  
not yet required

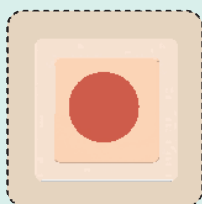

May wish to consider  
dressing change  
although not essential



Dressing change  
required when fluid is  
close to the edge of  
the pad

### Is the dressing showerproof?

Only bordered dressings are showerproof.

### Can creams/emollients be used under the dressing?


Creams and emollients should not be used under the dressing as it will stop the dressing being adhesive. You can however apply these to other surrounding skin.

### Can the dressing be cut?

The non-bordered dressing can be cut; however, your healthcare professional will have already selected the variation and size most appropriate to you and your wound.

### How should the dressing be stored?

It should be stored in a dry area at normal room temperature and kept away from sunlight.

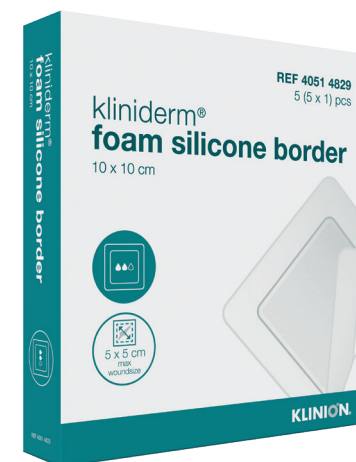

**This is a product overview and does NOT replace advice or guidance provided by your local Healthcare Professional and the local NHS protocols. If you have any further questions, please contact your Healthcare Professional.**

[www.kliniderm.co.uk](http://www.kliniderm.co.uk)

**h&r healthcare**  
A MEDIQ COMPANY

H&R Healthcare Ltd.  
3 Redcliff Road, Melton Park, Melton, Hull HU14 3RS  
T: +44 (0)1482 631606 E: [marketing@hrhealthcare.co.uk](mailto:marketing@hrhealthcare.co.uk)