

Devon™ Positioning Products



Available
on NHS SC

User guide

Devon™ Positioning Products offer a reliable and economical solution to pressure prevention

Key benefits

- For any area of the body that requires support stability and protection
- Reduces the risk of pressure ulcers whilst maintaining proper circulation
- Protection and support for the foot and heel, as well as other extremities that require support
- Can be used as part of a pressure ulcer prevention strategy
- Non-toxic, latex free and DEHP* free
- Fire retardant
- Lightweight, non-slip and simple to position
- Long lasting, disposable, single patient

Application instructions

Devon™ foot and heel protector

1. Ensure skin is intact or dressed according to local wound care protocols
2. Position for the most comfortable fit, ensuring the seam is not below the resting area of the foot with convoluted side up.
3. Fix in place using Velcro strap, ensure fit is comfortable without being restrictive
4. Monitor skin integrity in line with local guidelines
5. If deterioration in condition of the skin is noted, follow local guidelines and upgrade treatment plan according to pressure ulcer pathway.

Application instructions

Devon™ utility pad

1. Ensure skin is intact or dressed according to local wound care protocols
2. Position the Devon™ utility pad underneath area of the body that requires pressure redistribution - convoluted side up
3. Monitor skin integrity in line with local guidelines
4. If deterioration in condition of the skin is noted, follow local guidelines and upgrade treatment plan according to pressure ulcer pathway.

Frequently asked questions

Where can Devon™ positioning products be used?

Devon positioning products can be used on any area of the body that requires support, stability and protection (for the legs, ankles, heels, feet as well as elbows, knees and head). There are two products; Devon foot and heel protector and Devon utility pad – both designed to meet clinical needs.

Can I sit on the Devon utility pad?

Do not sit on the utility pad or use in isolation.

Which side of the Devon utility pad do I put against the skin?

The convoluted side should be placed against the skin.

How long can Devon positioning products be used for?

The Devon positioning products are indicated for single patient use and can be used continuously until they are no longer needed. However, if they become damaged or soiled they should be replaced.

How much pressure can be redistributed?

Devon positioning products are proven to reduce pressure distribution by 48% at levels above 32mmHg¹.

Can you use on very oedematous limbs?

Yes, however if you are concerned that Devon foot and heel protector may be snug consider Devon utility pad.

Can Devon positioning products be used in conjunction with each other?

Yes, you can use the Devon foot and heel protector and Devon utility pad at the same time to protect the heel, lower calf and behind the head.

Can Devon positioning products be used on multiple patients?

No, Devon positioning products are for single patient use only.

Can Devon positioning products be washed or cleaned?

No, Devon positioning products are not recommended for mechanical cleansing. Soiled or damaged products should be disposed of in accordance with your local clinical waste policy.

Can Devon positioning products be cut?


Yes, the Devon utility pad can be cut to suit clinical requirements.

Can a dressing still be worn with Devon positioning products?

Yes, it is important that appropriate wound care guidelines are adhered to as this forms part of a pressure ulcer prevention and treatment strategy.

What are Devon positioning products made out of?

Devon positioning products are moulded from non-toxic, latex-free foam.

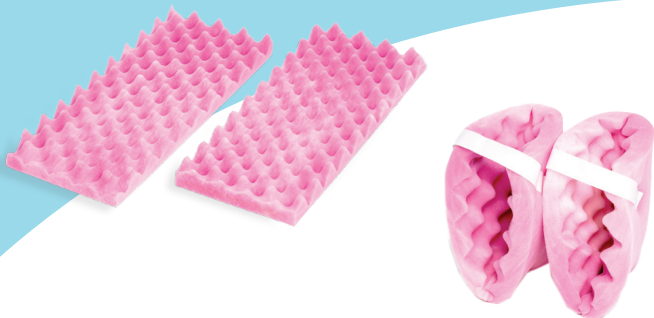

Ordering details

Item	Order quantity	Order code	PIP code	NHSSC code
Devon foot and heel protector	1 pair	FP-HEEL1CE	394-2018	FES10507
Devon utility pad	Pack of 2	FP-OVER1CE	394-2026	FES10508

Visit our H&R Healthcare YouTube channel for product application videos.

h&r healthcare
A MEDIQ COMPANY

H&R Healthcare Ltd.
3 Redcliff Road, Melton Park, Melton, Hull HU14 3RS
T: +44 (0)1482 631606 E: marketing@hrhealthcare.co.uk

*DEHP - di-(2-ethylhexyl)phthalate

References:

1. Covidien Study Morris V. Shelanski, M.D., C.M., S Holley, Operating Room Patient Positioning Pressure Distribution of Foam & Gel Devices, 2009.