
kliniderm® debride

INSTRUCTIONS FOR USE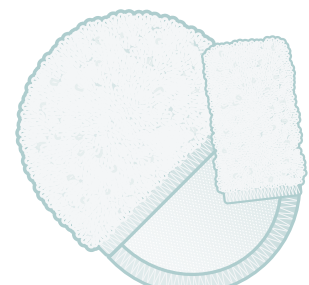

**KLINION®
WOUND CARE**

i

MD

CE
1282

STERILE

NO FIRE

NO SHARP

22
PAP

Farmaban S.A.
Carrer Vilella de Claret, 6
08272 Sant Fruitós de Bages
Barcelona
Spain

Medeco B.V.
Alexander Flemingstraat 2
NL-3261 MA Oud-Beijerland
The Netherlands

IFU_KWC059
0_040105
08-2021

www.medeco.org

1. Soak Kliniderm® debride with approximately 50-60ml (for Kliniderm® debride pocket use 10-20ml) of wound cleansing solution (e.g. saline or distilled water), follow local guidelines.

2. Applying gentle pressure, wipe the moistened side over the wound and/or surrounding skin in a circular motion.

3. Dispose according to local procedures and guidelines.

Precautions

- Take care when applying pressure or rubbing quickly and avoid causing the patient excessive pain.
- Ensure sterile application of the pad to the hand/finger, do not touch the wound contact side of the pad before use.
- Kliniderm® debride and Kliniderm® debride pocket are single-use. Use a new pad with every separate wound to prevent bacteria from spreading.
- Do not cut.
- Do not use as wound dressing (do not leave on the wound).
- Do not use on patient with a known sensitivity to one or more of the components of this product (polyester and polypropylene).
- Do not re-use. Re-use can cause serious infections.
- Do not re-sterilize.
- For external use only.
- For use under the guidance of a healthcare professional.
- Shelf life: 5 years.

1. 50-60 ml

2. 50-60 ml

1. 10-20 ml

2. 10-20 ml

ENGLISH

Product Description

Kliniderm® debride is a sterile monofilament polyester microfibrer pad with loops on the wound contact side. The inside is coated with polypropylene. The pad's design enables good grip and steering of the pad during use. Kliniderm® debride is intended as a rapid, highly effective, safe method of mechanical debridement. Used for absorbing exudate, cellular debris and keratosis during debridement and gentle on intact tissue.

MD Medical Device

Distributor

Indications

Kliniderm® debride is suitable for the treatment of superficial wounds (acute and chronic) and the surrounding skin of: diabetic foot ulcers, arterial and venous ulcers, pressure ulcers, post-operative wounds healing by secondary intention, lacerations and abrasions, burns/scalds, lymphoedema. The product is intended for adults with devitalized tissue caused by chronic and acute wounds in any healthcare setting. Kliniderm® debride and Kliniderm® debride pocket can be used interchangeably. Kliniderm® debride pocket is recommended when treating small superficial wounds or when extra control is needed when applying pressure. Kliniderm® debride is recommended when treating large-sized superficial wounds.

Contra-indications

No contraindications known to date.

Directions for use

- Soak Kliniderm® debride with approximately 50-60ml (for Kliniderm® debride pocket use 10-20ml) of wound cleansing solution (e.g. saline or distilled water), follow local guidelines.
- Applying gentle pressure, wipe the moistened side over the wound and/or surrounding skin in a circular motion.
- Dispose according to local procedures and guidelines.

Precautions

- Take care when applying pressure or rubbing quickly and avoid causing the patient excessive pain.
- Ensure sterile application of the pad to the hand/finger, do not touch the wound contact side of the pad before use.
- Kliniderm® debride and Kliniderm® debride pocket are single-use. Use a new pad with every separate wound to prevent bacteria from spreading.
- Do not cut.
- Do not use as wound dressing (do not leave on the wound).
- Do not use on patient with a known sensitivity to one or more of the components of this product (polyester and polypropylene).
- Do not re-use. Re-use can cause serious infections.
- Do not re-sterilize.
- For external use only.
- For use under the guidance of a healthcare professional.
- Shelf life: 5 years.

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